





HEDclip

HEDclip is uniquely created to help medical staff members fixate the blood tubes during a dialysis treatment. HEDclip fits easily onto a piece of clothing or a bed sheet and ensures a safe and hygienic fixation of the tubes.

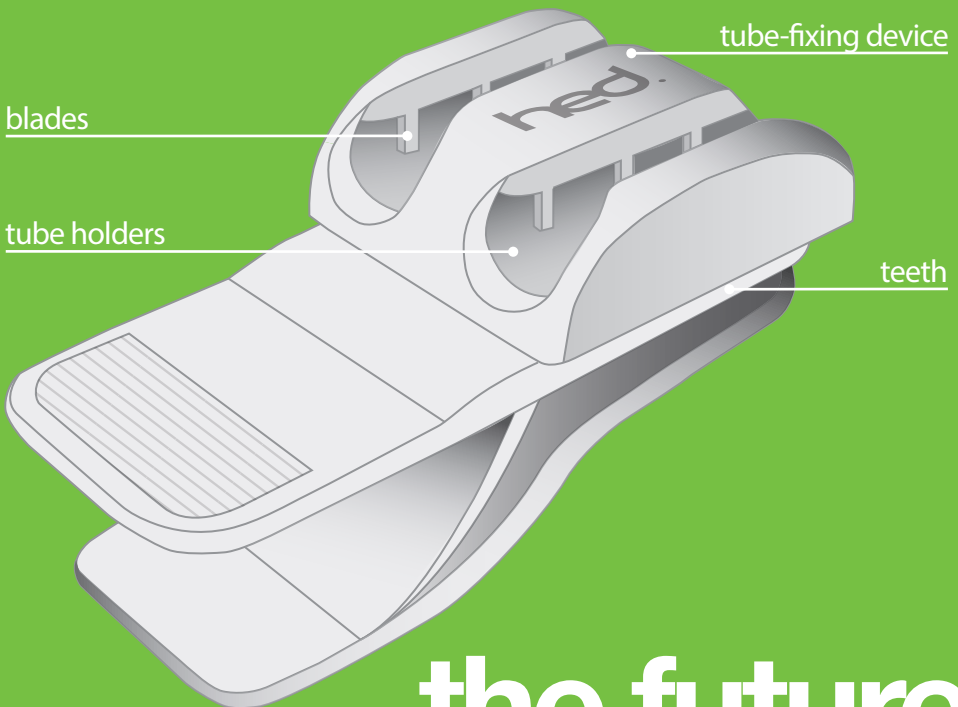

the future in fixation


User guide

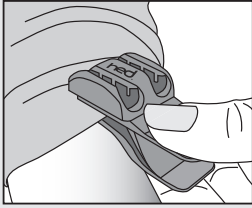

1. To place HEDclip


In order to place HEDclip on the textile in the best manner, you find an appropriate spot on the patient's clothes, on the bed sheet or where it might fit. Then open it by applying pressure on the bottom part and try to get as much clothing between the teeth as possible.

(On a plain piece of clothing it is important to catch a seam when placing the product)

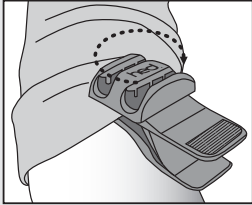


2. Placement of the top

Place the tube-fixing device so the tube holders are parallel to the needles/fistulas. This function enables the bottom part to be placed in whichever angle is preferred, while still maintaining the right angle of the tube-fixation device in relation to the needles/fistulas.

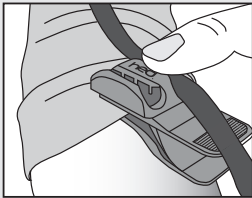

3. Fitting of the tubes

When HEDclip is placed in the right manner, the tubes are to be fitted into the tube holders. Slide the tubes down into the holders by applying pressure on one tube at the time in a sliding motion.

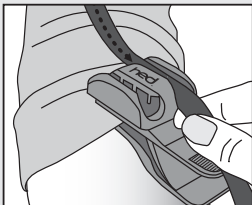

4. Adjusting the tubes

If it is necessary to adjust the tubes, simply pull them slowly in the direction you wish.

The small blades in the tube holders are made to prevent that too much force can be applied to the needles/fistulas.

Cautions

Cleaning:

HEDclip does not require sterilization and can be cleaned with aqueous or hydro-alcoholic solutions with cleanser (e.g. chlorine dioxide, peracetic acid, glutaraldehyde, benzalkonium chloride) However, if it comes in contact with blood or becomes soiled, it can be cleaned at temperatures from 100°C up to 130°C. (NB! Frequent cleaning at high temperatures will shorten the life expectancy of the product).

Handling and storage:

Ensure that HEDclip is not exposed to direct sunlight when not in use. This will shorten HEDclip's life expectancy and diminish its durability.

Ensure that HEDclip is not stored in a mechanically stressful position (i.e., jaw open) when not in use. This diminishes its spring and will temporarily or permanently reduce its capability to attach to clothing.

Warranty and limitation of liability

1. The Manufacturer warrants its products to be free from defects in material and workmanship for a period of 3 years from the date of purchase. This warranty does not apply to normal wear.

2. The Manufacturer shall not be liable for any misuse, improper handling, non-compliance with warnings and instructions, damage arising from events after the Manufacturer's release of HEDclip and failure or omission to inspect the product before use in order to insure that it is in proper condition.

3. The Manufacturer is Health Equipment Denmark, Skt. Anna Gade 45a 3.th, 8000 Aarhus C, Denmark.